# Freestyle™

*No.1 choice  
of hospitals  
& mothers*

EN Instructions for use | ES Manual de instrucciones  
PT Instruções de utilização | EL Οδηγίες χρήσης  
HE הוראות שימוש

Thank you for choosing the Freestyle breast pump. Breast milk is the natural choice for your baby, but there are reasons and situations where breastfeeding is not possible. For over 50 years, it has been Medela's conviction that these mothers and babies can be supported. In close cooperation with breastfeeding experts, Medela, one of the market leaders, has researched and developed breast pumps that perfectly meet breastfeeding mothers needs. For mothers who insist on the best for their babies and themselves. Because you care.

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# Table of Contents

4

<b>1. Intended use/Population - Contraindication</b>	<b>5</b>
<b>2. Meaning of symbols</b>	<b>6</b>
<b>3. Important safety information</b>	<b>8</b>
<b>4. Product description</b>	<b>10</b>
4.1 Freestyle – all-in-one, innovative, time-saving	10
4.2 Product parts	10
<b>5. Cleaning</b>	<b>12</b>
5.1 Before using for the first time and after every use	12
5.2 Before using for the first time and once a day	14
5.3 Cleaning the motor unit	15
5.4 Cleaning – summary	16
<b>6. Preparing for pumping</b>	<b>18</b>
6.1 Mains operation	18
6.2 Battery operation	19
6.3 Assembling the pump set	20
<b>7. Expression</b>	<b>22</b>
7.1 Preparing for single pumping	22
7.2 Pumping	22
7.3 Preparing for hands-free expression with Easy Expression Bustier	26
7.4 Preparing for double pumping	26
<b>8. Storing and thawing breast milk</b>	<b>28</b>
8.1 Storage	28
8.2 Freezing	28
8.3 Thawing	28
<b>9. Feeding breast milk</b>	<b>29</b>
<b>10. Troubleshooting</b>	<b>30</b>
<b>11. Warranty/disposal</b>	<b>31</b>
<b>12. Complete care during breastfeeding time</b>	<b>32</b>
<b>13. EMC/Technical description</b>	<b>154</b>
<b>14. Technical specifications</b>	<b>160</b>

# 1. Intended use / Population – Contraindication

## **The intended use and indications of breast pumps**

Breast pumps are intended to be used by lactating women in the hospital or home environment for expressing breast milk. Breast pumps are indicated to alleviate the symptoms of engorgement by expressing milk. In case of mastitis breast pumps support the healing process by removing breast milk from the affected breast. Breast pumps are indicated to relieve sore and cracked nipples and to bring out flat or inverted nipples. Also breast pumps enable mothers to provide breast milk for babies who cannot feed directly at the breast for a variety of reasons such as latch-on problems, cleft palate or premature babies.

## **Intended patient population/user**

Adolescent paediatric or adult lactating women. Many women find it convenient, or even necessary, to use a breast pump to express and store their breast milk once they have returned to work, are travelling or are otherwise separated from their baby. Women are often delighted to learn that a breast pump can be used as a supplement to breastfeeding and that some pumps are designed to mimic the suckling of a nursing baby.

## **Contraindications**

There are no known contraindications for the Freestyle breast pump.

## 2. Meaning of symbols

6

The warning symbol identifies all instructions that are important for safety. Failure to observe these instructions can lead to injury or damage to the breast pump! When used in conjunction with the following words, the warning symbol stands for:

**Warning**  
Can lead to serious injury or death.

**Note**  
Can lead to material damage.

**Caution**  
Can lead to minor injury.

**Info**  
Useful or important information that is not related to safety.

### Symbols on the packaging

This symbol indicates that the material is part of a recovery/recycling process.

This symbol indicates a carton package.

This symbol indicates to keep the device away from sunlight.

This symbol indicates to handle the fragile device with care.

This symbol indicates the temperature limitation for operation, transport and storage.

This symbol indicates the humidity limitation for operation, transport and storage.

This symbol indicates the atmospheric pressure limitation for operation, transport and storage.

This symbol indicates to keep the device dry.

This symbol indicates that the packaging contains products intended to come in contact with food according to regulation 1935/2004.

This symbol indicates unique GSI Global Trade Item Numbers (GTIN).

This symbol indicates do not dispose the device together with unsorted municipal waste (for EU only).

This symbol indicates to follow instructions for use.

### Symbols on the device

This symbol indicates to follow instructions for use.

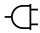
This symbol indicates the compliance with the essential requirements of the Council Directive 93/42/EEC of 14 June 1993 concerning medical devices.


This symbol indicates the manufacturer.

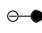This symbol indicates where to open the battery cover.

-  This symbol indicates where to plug in the mains adapter.
-  This symbol indicates do not dispose the device together with unsorted municipal waste (for EU only).
-  This symbol indicates the compliance with additional USA and Canada safety requirements for medical electrical equipment.
-  This symbol indicates a type BF applied part.
- REF** This symbol indicates manufacturer's catalogue number of the battery.
- SN** This symbol indicates manufacturer's serial number of the device.
- IP22** This symbol indicates the protection against ingress of solid foreign objects and against harmful effects due to the ingress of water.
-  This symbol indicates the date of manufacture (four digits for the year and two digits for the month).

## Symbols on the mains adapter

-  This symbol indicates that the mains adapter is a class II device.
-  This symbol indicates the compliance with China Compulsory Certification.
-  This symbol indicates that the mains adapter is safety tested.
-  This symbol indicates that the mains adapter is for indoor use only.
-  This symbol indicates the compliance with USA and Canada safety requirements.
-  This symbol indicates the compliance with Federal Communications Commission requirements.
-  This symbol indicates polarity of d.c. power connector.
-  This symbol indicates the compliance with AUS/NZ regulatory requirements (Regulatory compliance mark).
-  This CE-mark indicates the compliance with the low voltage and electromagnetic compatibility directive.
-  This symbol indicates to not dispose the device together with unsorted municipal waste (for EU only). The solid bar symbol indicates that mains adapter is put on the market after 13 August 2005.
-  This symbol indicates the compliance with Japanese safety requirements.
-  This symbol indicates the compliance with energy efficiency requirements.
-  This symbol indicates alternating current.
-  This symbol indicates direct current.

## 3. Important safety information

8

Failure to follow these instructions/safety information can lead to danger from the device. Subject to technical changes.

### Device

Danger of electrocution! Keep the device dry! Never immerse in water or other liquids!

The Freestyle breast pump is not heat-resistant: keep away from radiators and open flames.

Do not expose the motor unit to direct sunlight.

Repairs must be performed only by an authorised service agency. Do not repair yourself! No modifications to the device are permitted.

Never use a damaged device. Replace damaged or worn parts.

The operating life of the Freestyle breast pump is 250 hours. The service life is two years.

### Electrical use

Separation from the mains is only assured through the disconnection of the mains adapter from the wall socket outlet.

Keep the mains adapter away from heated surfaces.

Do not reach for any electrical device if it has fallen into water. Unplug immediately.

The Freestyle breast pump should never be left unattended when plugged into a power source.

## Use

Use the Freestyle breast pump only for its intended use as described in this manual.

Never operate an electrical device if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.

Do not use the Freestyle breast pump while bathing or showering.

Do not use the Freestyle breast pump while sleeping or overly drowsy.

This is a single user product. Use by more than one person may present a health risk.

Do not drive whilst hands-free pumping.

Supervision is necessary when the Freestyle breast pump is used in the vicinity of children.

Contact your lactation consultant or physician if you experience problems or pain.

Portable and mobile radio frequency communications equipment can affect the Freestyle breast pump.

## Important:

- I Plastic bottles and component parts become brittle when frozen and may break when dropped.
- I Also, bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over.
- I Take appropriate care in handling bottles and components.
- I Do not use the breast milk if bottles or components become damaged.

## 4. Product description

10

### 4.1 Freestyle - all-in-one, innovative, time-saving

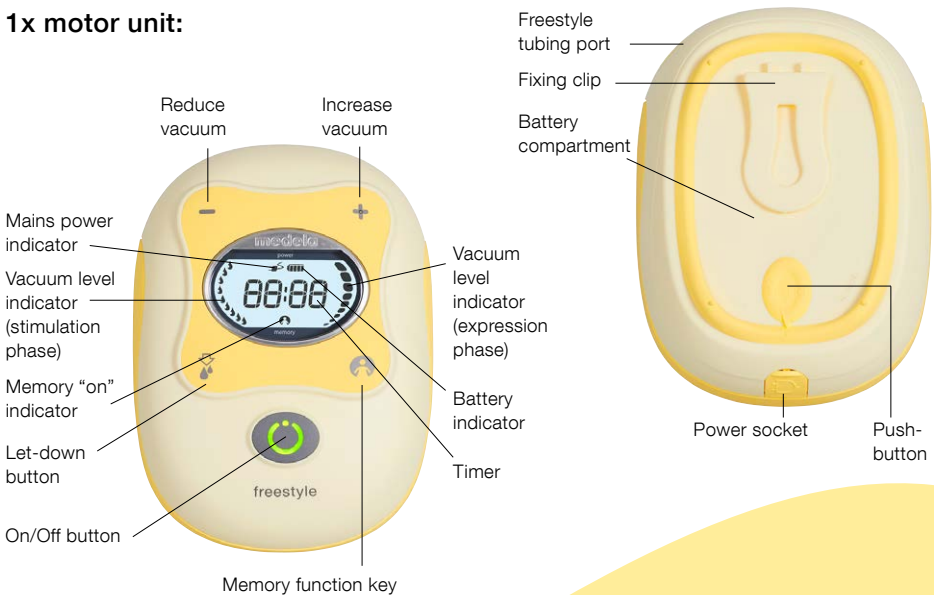
By buying Freestyle, you have made the right choice. Freestyle is both efficient and comfortable. Freestyle is **efficient** because it uses 2-Phase Expression pumping technology, which mimics the baby's natural sucking rhythm. Short, stimulating patterns are followed by longer ones, ensuring an optimal flow of milk. Combined with double pumping, this helps to increase and maintain milk production and saves precious time. Freestyle is **comfortable** thanks to its adjustable vacuum settings, memory function and information display, which are very convenient. All this makes Freestyle ideal for working mums and for daily and regular use.

#### The main features of your Freestyle breast pump are:


- I Helps to maintain and increase milk production and saves time;
- I Is small, lightweight and comfortable;
- I Offers maximum mobility and freedom of movement.

### 4.2 Product parts

#### 1x motor unit:

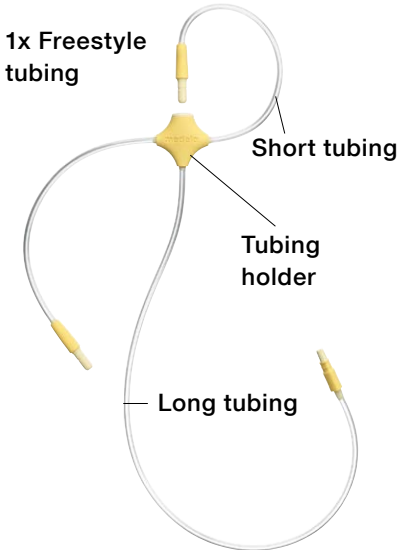Spare parts and accessories available for order can be found in chapter 12.


**2x PersonalFit™ breast shield M (24 mm)**  
(Applied part)

(for other breast shield sizes see chapter 12)

**2x Freestyle connectors**

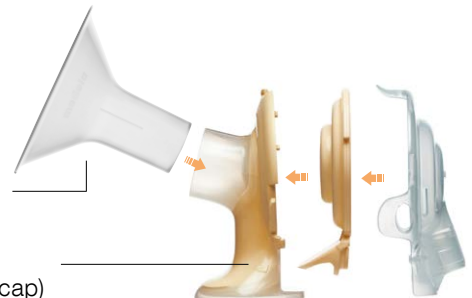(breast shield base, membrane, transparent cap)

1x rechargeable battery

## 5. Cleaning

12

### Caution

- | Only use drinking-quality water for cleaning.
- | Take apart and wash all parts that come in contact with the breast and breast milk immediately after use to avoid dry up of milk residues and to prevent growth of bacteria.
- | Never dry parts in a microwave, as this may destroy them.

### Note

- | The pump set requires no maintenance.
- | Take care not to damage parts of the pump set during cleaning.
- | If the individual components of the pump set are cleaned in a dishwasher parts may be discoloured by food pigments. This has no impact on the function.

### 5.1 Before using for the first time and after every use

Disassemble the pump set into individual parts.

Rinse all parts with cold, clear water (approx. 20 °C) for 10–15 seconds.

2

Rinse all parts with cold, clear water (approx. 20 °C).

3

Clean all parts with plenty of warm soapy water (approx. 30 °C).

5

Dry with a clean cloth or leave to dry on a clean cloth.

or

As an alternative to steps 2, 3 and 4.

If you clean the individual parts of the pump set in the dishwasher, place them on the top rack or in the cutlery section.

## 5. Cleaning

14

### Caution

- I Only use drinking-quality water for cleaning.
- I Take apart and wash all parts that come in contact with the breast and breast milk immediately after use to avoid dry up of milk residues and to prevent growth of bacteria.
- I Never dry parts in a microwave, as this may destroy them.

### Note

- I When boiling, a tea spoon of citric acid may be added to avoid a build-up of limescale.
- I Store the pump set in a clean bag/ container until next use. Or store it in clean paper or a cloth.

### 5.2 Before using for the first time and once a day

Disassemble the pump set into individual parts.

Dry with a clean cloth or leave to dry on a clean cloth.

2a

or

2b

Cover all parts with water and boil for five minutes.

Use QuickClean\* bags in the microwave in accordance with the instructions.

### 5.3 Cleaning the motor unit

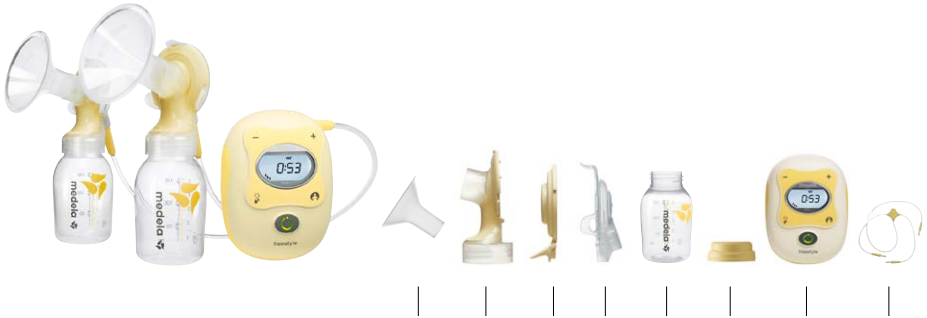
1

Wipe off with a clean, damp cloth.

# 5. Cleaning

## 5.4 Cleaning – summary

	1	2	3	4	5	6	7	8
<b>Before using for the first time and after every use</b>								
Rinse with cold, clear water (approx. 20 °C), then clean with plenty of warm soapy water (approx. 30 °C) and finally rinse with cold, clear water for 10–15 seconds;	✓	✓	✓	✓	✓	✓	--	--*
Or wash in the dishwasher (on the top rack or in the cutlery section).	✓	✓	✓	✓	✓	✓	--	--*
<b>Before using for the first time and once a day</b>								
Cover with water and boil for five minutes;	✓	✓	✓	✓	✓	✓	--	--*
Or use a Quick Clean Microwave bag in accordance with the instructions;	✓	✓	✓	✓	✓	✓	--	--*
Or wipe off with a clean, damp cloth.	--	--	--	--	--	--	✓	--*

\*For more information, see chapter 10 Troubleshooting ("If there is low or no suction")

## 6. Preparing for pumping

18

### Warning

#### re 6.1 Mains operation:

- I Follow the **exact** order of the steps.
- I Only use the mains adapter that comes with Freestyle.
- I Make sure the voltage of the mains adapter is compatible with the power source.

### Note

- I Do not use force to push the battery into the battery compartment.
- I Do not insert the battery while the pump is on mains.

### Info

- I You can operate the Freestyle breast pump via battery or mains. If you want to use the battery, you must charge it before first use.

#### re 6.2 Battery operation:

- I The battery starts charging as soon as you connect the pump to a power source.
- I You can use the pump during charging.
- I Do not disconnect the mains adapter until charging is completed.
- I If possible, do not remove the battery from the motor unit. Removing the battery requires a recharging time of up to 24 hours.
- I Do you plan to store the pump for an extended period of time? Then it is better to remove the battery from the unit.
- I If Freestyle has been inactive for 30 minutes, it automatically switches off.

## 6.1 Mains operation

To operate the pump via mains power, uncover the power socket on the bottom of the motor unit. Insert the power adapter (the small round plug) into the socket.

Insert the battery into the battery compartment with the “This Side Down” arrows pointing towards the compartment. The gold contacts on the battery must touch the contacts in the compartment. Finally, put the battery cover back on.

## 6.2 Battery operation

2

For information on assembling the pump set, see chapter 6.3.

Slide the country-specific plug onto the larger adapter unit (if this is not already done so). Plug the plug into the mains socket on the wall. The motor unit is now ready for operation.

1

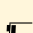
To operate the pump via battery, open the battery compartment by pressing the push-button on the battery cover (on the back of the motor unit) and sliding the cover in the direction of the arrow.

3

More information on mains operation in chapter 6.1.

Charge the battery by connecting the motor unit to the mains. "bAtt" will appear on the display on the motor unit's front. Charge the battery until the battery indicator on the display stops flashing and "bAtt" disappears. The battery is now fully charged. Disconnect the mains adapter. The pump is ready for operation via battery.

	Mains adapter connected to socket
 flashing	Battery charge status
	Battery fully charged, approx. 3 hours' expression
 flashing	20 min. expression time remaining.
 flashing	Battery flat
"bAtt"	Initial charge and charging after removing the battery (up to 24 hours charging time)

The above table shows the different operation statuses. These symbols appear on the display on the front of the motor unit.

## 6. Preparing for pumping

20

### Caution

I Wash hands thoroughly (at least one minute) with soap and water before touching the breast and pump parts.

### Note

- I Use Medela original accessories only.
- I Check pump set components for wear or damage before use and replace if necessary.
- I All components must be completely dry before use.

### Info

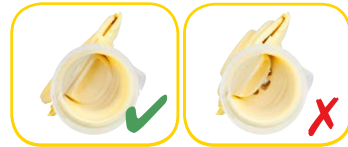
I Carry out all the steps with care and assemble the pump set correctly. Otherwise a good vacuum may not be achieved.

### 6.3 Assembling the pump set

1

Put the yellow membrane onto the base of the breast shield connector:

View from below

4

Screw the bottle onto the connector.

→ To assemble the second breast shield/bottle/connector combination for double pumping, repeat steps 1-4.

2

Click the transparent cap onto the base/membrane combination.

→ Check that the cap is firmly connected to all three connection points (one at the top, two at the sides).

3

Push the breast shield onto the connector.

5

Insert the short tubing into the opening in the transparent cap. Push it inside as far as it will go.

6

Insert the long tubing into the tubing port at the top of the motor unit.

## 7. Expression

22

### Caution

#### 1 re step 2 (chapter 7.1):

Answer the following questions to find out whether the available breast shield size will fit:

- | Is your nipple moving freely in the tunnel?
- | Is minimal or no areolar tissue being pulled into the tunnel of the breast shield?
- | Do you see a gentle, rhythmical motion in the breast with each cycle of the pump?
- | Do you feel the breast emptying all over?
- | Is your nipple pain-free?

If the answer to any of these questions is “No” or if you experience problems or pain during expression, consult your breastfeeding and lactation consultant or midwife.

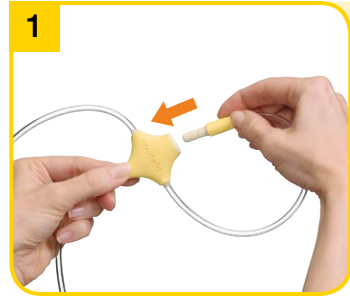
### Note

#### 1 re step 3 (chapter 7.1):

Do not hold the pump set by the bottle. This can lead to blockage of the milk ducts and engorgement.

- | Wipe the breast with a warm washcloth (do not use alcohol).
- | Restart timer: press  for 3 seconds. Timer is restarted but Freestyle continues to pump.

### 7.1 Preparing for single pumping

Insert the unused short tubing into the tubing holder.

### 7.2 Pumping

Switch on the breast pump with

- Timer starts.
- The stimulation phase lasts a maximum of two minutes. The pump then switches off automatically in expression phase.

2

Place the breast shield on your breast so that your nipple is properly **centred** in the tunnel.

3

Hold the breast shield onto your breast with your thumb and index finger. Support your breast with the palm of your hand.

2

If your milk starts to flow earlier, press  to change to the expression phase.

→ Make sure that the milk is flowing into the bottle properly.

3

The vacuum can be adjusted in every phase. Comfort vacuum: Use  to increase the vacuum until it is slightly unpleasant. Then press  to dial it back minimally.

## 7. Expression

24

### Caution

I Contact your midwife or lactation consultant if you can express only minimal or no milk or if expression is painful.

### Note

I Always disconnect Freestyle from the mains after expression (except if the battery is charging).

I Only fill the bottle to the 150 ml mark.

### Info

I Freestyle switches off automatically if it has been running for 30 minutes with no manipulations.

I Do not kink the tubing during expression.

### I re step 4:

Memory function key 

To activate your personal setting, start the pump and press .

4

Press  for three seconds during the expression phase to save the individual pumping programme.

→ The indicator on the display flashes and the pump emits three beeps.

→ Finish expression as usual.

7

Clean in accordance with Chapter 5.

Close the bottle with a lid.

→ Follow the instructions in Chapter 8 “Storing and thawing breast milk”.

5

Switch off the breast pump with  .

6

Use the stand to prevent the bottle from tipping over.

## 7. Expression

26

### Caution

#### Important step 2:

Do not hold the pump set by the bottle. This can lead to blockage of the milk ducts and engorgement.

### Info

Double pumping saves time and increases the energy content of the milk. Milk production is increased and maintained long-term.

### 7.3 Preparing for hands-free expression with the Easy Expression Bustier\*

1

Put on the Easy Expression Bustier and leave the zip open slightly.

### 7.4 Preparing for double pumping

1

Place the breast shield on your breast so that your nipple is properly **centred** in the tunnel. Hold the breast shield onto your breast with your thumb and index finger. Support your breast with the palm of your hand.

2

Place the breast shield on your breast under the bustier such that your nipple is **centred** in the tunnel of the breast shield.

3

Close the zip and insert the connector onto the breast shield.

\*The Easy Expression Bustier is **not** included with the Freestyle breast pump. See chapter 12 for ordering details.

2

Switch on the breast pump with .

- Timer starts.
- The stimulation phase lasts a maximum of two minutes. The pump then switches off automatically in expression phase.

3

Place the second breast shield on your breast so that your nipple is properly **centred** in the tunnel.

## 8. Storing and thawing breast milk

28

### 8.1 Storage

#### Storage guidelines for freshly expressed breast milk

(For healthy term babies)

Room temperature	Refrigerator	Freezer	Thawed breast milk
4–6 hours at 19 to 26 °C (66 to 78 °F)	3–8 days at 4 °C (39 °F) or lower	6–12 months –18 to –20 °C (0 to –4 °F)	In refrigerator for 10 h max. Do not refreeze!

I Do not store breast milk in the door of the refrigerator. Instead choose the coldest part of the refrigerator (this is at the back of the glass shelf above the vegetable compartment).

### 8.2 Freezing

I You can freeze expressed breast milk in milk bottles or “Pump & Save” bags. Do not fill the bottles or bags more than  $\frac{3}{4}$  full to allow space for possible expansion.

I Label the bottles or “Pump & Save” bags with the date of expression.

### 8.3 Thawing

#### Warning

Do not thaw frozen breast milk or do not warm breast milk in a microwave or a pan of boiling water to avoid loss of vitamins, minerals and other important components and to prevent burns.

I In order to preserve the components of the breast milk, thaw the milk in the refrigerator overnight. Alternatively, you can hold the bottle or “Pump & Save” bag under warm water (max. 37 °C).

I Gently swirl the bottle or “Pump & Save” bag to mix any fat that has separated. Avoid shaking or stirring the milk.

## 9. Feeding breast milk

### Info

Medela recommends Calma for feeding breast milk.

#### With Calma

Your baby can feed, pause and breathe regularly.

Milk will flow when a vacuum is created.

Your baby's natural feeding behaviour will be maintained, supporting easy transition back to the breast.

Calma – unique feeding solution for breast milk.

You can find detailed information about Calma on the Medela website ([www.medela.com](http://www.medela.com)).

As a result of its comprehensive research work, Medela is one of the first companies in the world to incorporate babies' breastfeeding behaviour into its 2-Phase Expression and Calma innovations. 2-Phase Expression mimics the baby's natural sucking rhythm to express more milk in less time. Short, stimulating patterns are followed by longer ones, providing optimal milk flow. The unique Calma breast milk feeding device allows the baby to pause and breathe during feeding whilst maintaining the suckling behaviour learned at the breast. Developed for a great breastfeeding experience.

## 10. Troubleshooting

30

Problem	Solution
<b>If there is low or no suction</b>	<p>Check all connections.</p> <p>Check that the transparent cap is firmly connected to all three connection points on the connector between breast shield and bottle.</p> <p>Hold the breast shield, not the connector or bottle, while expressing.</p> <p>Ensure the breast shield forms a complete seal around the breast.</p> <p>During single pumping, insert the unused tubing into the tubing holder. Do not kink the tubing.</p> <p>There should be no water in the tubing.</p> <p>All components must be completely dry.</p>
<b>The pump becomes wet</b>	<p>Unplug the mains adapter from the socket.</p> <p>Switch off the pump.</p> <p>Do not place the pump upside down. Control panel must point upwards.</p> <p>Store the pump in a dry, warm place for 24 hours.</p> <p><b>Note:</b></p> <p>Do not expose the pump to direct sunlight.</p> <p>Do not accelerate the drying process by use of heat.</p>
<b>“Err” + 3 beeps</b>	<p>Contact your Medela dealer or the store where you bought the pump.</p>
<b>The mains adapter symbol flashes The pump does not switch on</b>	<p>Contact your Medela dealer or the store where you bought the pump.</p>
<b>The battery indicator flashes with four bars</b>	<p>Contact your Medela dealer or the store where you bought the pump.</p>

# 11. Warranty / disposal

## Warranty

The "International Warranty" brochure contains detailed warranty information.

## Disposal

The unit is made of various metal and plastics. Before disposal, the device is to be rendered unusable and it must not be disposed of as unsorted municipal waste in accordance with local regulations. Use your local return and collection system for waste electrical and electronic equipment (incl. batteries). Improper disposal may have harmful effects on the environment and on public health.

## 12. Complete care during breastfeeding time

32

The following accessories for the Freestyle are available from Medela distribution partners by specifying the product number.

### Spare parts

Article number	Product
<b>099.0271</b>	Freestyle Adapter (100-240V; 12V) Euro
<b>099.0272</b>	Freestyle Adapter (100-240V; 12V) UK and US / JP
<b>099.0273</b>	Freestyle Adapter (100-240V; 12V) AUS / NZ
<b>099.0274</b>	Freestyle Adapter (100-240V; 12V) CN
<b>200.1690</b>	Freestyle Motor Unit
<b>200.1692</b>	Freestyle Tubing
<b>200.1691</b>	Freestyle Rechargeable Battery

### Accessories

Article number	Product
<b>008.0332*</b>	PersonalFit breast shield S (21 mm)
<b>008.0333*</b>	PersonalFit breast shield M (24 mm)
<b>008.0334*</b>	PersonalFit breast shield L (27 mm)
<b>008.0335*</b>	PersonalFit breast shield XL (30 mm)
<b>008.0336*</b>	PersonalFit breast shield XXL (36 mm)
<b>800.0797</b>	Breast milk Bottle 150 ml (3 pcs)
<b>008.0137</b>	Breast milk Bottle 250 ml (2 pcs)
<b>008.0141</b>	Calma Solitaire
<b>008.0316</b>	Pump & Save breast milk Bags
<b>008.0217</b>	Easy Expression Bustier white S
<b>008.0220</b>	Easy Expression Bustier white M
<b>008.0224</b>	Easy Expression Bustier white L
<b>008.0225</b>	Easy Expression Bustier white XL

Other Medela products are available at [www.medela.com](http://www.medela.com)

\* If the result of expression is unsatisfactory or expression is painful, contact your midwife or lactation consultant. A different size of PersonalFit™ breast shield may help make expression more comfortable and successful.

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## 13. EMC/Technical description

The Freestyle breast pump needs special precautions regarding EMC (Electromagnetic compatibility) and needs to be installed and put into service according to the EMC information provided in these instructions for use. Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies can affect the electric breast pump Freestyle and should be kept at least a distance 1.0 m away from the equipment. Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 1)

Electromagnetic emissions		
The electric breast pump Freestyle is intended for use in the electromagnetic environment specified below. The customer or the user of the electric breast pump Freestyle should assure that it is used in such an environment.		
Emission tests	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The electric breast pump Freestyle uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The electric breast pump Freestyle is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable <sup>a</sup>	
<sup>a</sup> No significant voltage changes		

 **Warning** – The electric breast pump Freestyle should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the electric breast pump Freestyle should be observed to verify normal operation in the configuration in which it will be used.

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 2)

<b>Electromagnetic immunity</b>			
<p>The electric breast pump Freestyle is intended for use in the electromagnetic environment specified below. The customer or the user of the electric breast pump Freestyle should assure that it is used in such an environment.</p>			
<b>Immunity test</b>	<b>IEC 60601 test level</b>	<b>Compliance level</b>	<b>Electromagnetic environment – guidance</b>
Electrostatic discharge ( ESD ) IEC 61000-4-2	$\pm 6$ kV contact $\pm 8$ kV air	$\pm 6$ kV contact $\pm 8$ kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Electrical fast transient / burst IEC 61000-4-4	$\pm 2$ kV for power supply lines $\pm 1$ kV for input / output lines	$\pm 2$ kV for power supply lines Not applicable <sup>a</sup>	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	$\pm 1$ kV line( s ) to line(s) $\pm 2$ kV line( s ) to earth	$\pm 1$ kV differential mode Not applicable <sup>b</sup>	Mains power quality should be that of a typical commercial or hospital environment.
<sup>a</sup> No input/output lines <sup>b</sup> No earth, class II equipment			

## 13. EMC/Technical description

<p>Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11</p>	<p>&lt;5 % <math>U_T</math> (&gt;95 % dip in <math>U_T</math>) for 0,5 cycle 40 % <math>U_T</math> (60 % dip in <math>U_T</math>) for 5 cycles 70 % <math>U_T</math> (30 % dip in <math>U_T</math>) for 25 cycles &lt;5 % <math>U_T</math> (&gt;95 % dip in <math>U_T</math>) for 5 s</p>	<p>&lt;5 % <math>U_T</math> (&gt;95 % dip in <math>U_T</math>) for 0,5 cycle 40 % <math>U_T</math> (60 % dip in <math>U_T</math>) for 5 cycles 70 % <math>U_T</math> (30 % dip in <math>U_T</math>) for 25 cycles &lt;5 % <math>U_T</math> (&gt;95 % dip in <math>U_T</math>) for 5 s</p>	<p>Mains power quality should be that of a typical commercial or hospital environment. If the user of the electric breast pump Freestyle requires continued operation during power mains interruptions, it is recommended that the electric breast pump Freestyle is powered from an uninterruptible power supply or a battery.</p>
<p>Power frequency (50/60 Hz) magnetic field IEC 61000-4-8</p>	<p>3 A/m</p>	<p>3 A/m</p>	<p>Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.</p>

NOTE  $U_T$  is the a.c. mains voltage prior to application of the test level.

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 4)

Electromagnetic immunity			
The electric breast pump Freestyle is intended for use in the electromagnetic environment specified below. The customer or the user of the electric breast pump Freestyle should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 Vrms	Portable and mobile RF communications equipment should be used no closer to any part of the electric breast pump Freestyle, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.  Recommended separation distance $d = 1,2\sqrt{P}$
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,5 GHz	3 V/m	$d = 1,2\sqrt{P}$ 80 MHz to 800 MHz $d = 2,3\sqrt{P}$ 800 MHz to 2,5 GHz where $P$ is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and $d$ is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>a</sup> should be less than the compliance level in each frequency range. <sup>b</sup> Interference may occur in the vicinity of equipment marked with the following symbol: 

## 13. EMC/Technical description

Note 1 At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

<sup>a</sup> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the electric breast pump Freestyle is used exceeds the applicable RF compliance level above, the electric breast pump Freestyle should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the electric breast pump Freestyle.

<sup>b</sup> Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Electromagnetic Compatibility (EMC, IEC 60601-1-2:2007, Table 6)

**Recommended separation distances between portable and mobile RF communications equipment and the electric breast pump Freestyle**

The electric breast pump Freestyle is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the electric breast pump Freestyle can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the electric breast pump Freestyle as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter M		
	150 kHz to 80 MHz $d = 1,2\sqrt{P}$	80 MHz to 800 MHz $d = 1,2\sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,37	0,37	0,74
1	1,2	1,2	2,3
10	3,7	3,7	7,4
100	12	12	23

For transmitters rated at a maximum output power not listed above, the recommended separation distance  $d$  in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where  $P$  is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

Note 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

# 14. Technical specifications

vacuum (approx.)  
-20....-270 mmHg  
-3....-35 kPa  
45 ... 120 cpm

In	Out
100-240V- 50/60 Hz	12V  1 A / 1.5 A

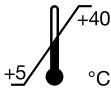
7.2V, 2250mAh  
Li-Ion


122 x 90 x 58

370 g   
270 g

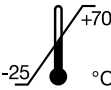Operation  
Funcionamiento  
Funcionamento  
Λειτουργία  
הפעלה

Transport / Storage  
Transporte y Conservación  
Transporte / Armazenamento  
Μεταφορά / Αποθήκευση  
הובלה/גזירה

Operation  
Funcionamiento  
Funcionamento  
Λειτουργία  
הפעלה

Transport / Storage  
Transporte y Conservación  
Transporte / Armazenamento  
Μεταφορά / Αποθήκευση  
הובלה/גזירה

Transport/Storage/Operation  
Transporte/Conservación/Funcionamiento  
Transporte / Armazenamento/Funcionamento  
Μεταφορά / Αποθήκευση/Λειτουργία  
הובלה/גזירה/הפעלה

kPa

 Medela AG  
Lättichstrasse 4b  
6341 Baar, Switzerland  
[www.medela.com](http://www.medela.com)

**International Sales**

Medela AG  
Lättichstrasse 4b  
6341 Baar  
Switzerland  
Phone +41 41 562 51 51  
Fax +41 41 562 51 00  
[www.medela.com](http://www.medela.com)

**Spain & Portugal**

Productos Medicinales Medela, S.L.  
C/ Llacuna, 22 4ª Planta  
08005, Barcelona  
Spain  
Phone +34 93 320 59 69  
Fax +34 93 320 55 31  
[info@medela.es](mailto:info@medela.es)  
[www.medela.es](http://www.medela.es)  
Portugal  
Phone 808 203 238  
[info@medela.pt](mailto:info@medela.pt)  
[www.medela.pt](http://www.medela.pt)