Apgar Timer

- EN Instructions for use
- DE Gebrauchsanweisung
- FR Mode d'emploi
- IT Istruzioni per l'uso
- NL Gebruiksaanwijzing
- SV Bruksanvisning
- DA Brugsanvisning
- FI Käyttöohjeet
- NO Bruksanvisning
- ES Instrucciones de uso
- PT Instruções de utilização
- EL Εγχειρίδιο χρήσης

Table of Contents

| | | |
|----|------------------------------------|-----|
| 1 | Intended use..... | 3 |
| 2 | Contraindications..... | 3 |
| 3 | Safety instructions | 3 |
| 4 | Operating elements | 4 |
| 5 | Operation of the Apgar Timer | 5 |
| 6 | Cleaning | 7 |
| 7 | Disposal | 7 |
| 8 | Warranty..... | 8 |
| 9 | Technical Data..... | 9 |
| 10 | Signs and symbols | 10 |
| 11 | Technical Documentation..... | 126 |

1 Intended use

The Medela Apgar Timer is used during childbirth in the hospital to measure and indicate a length of time at predetermined intervals from the setting time; this being when the baby is born. The Apgar score is used to assist a healthcare professional to determine the physical health of a newborn infant and its response to birth and life outside the womb at intervals of 1, 5 and 10 minutes. The total numerical score is 10. In addition, the exact time of birth can be stored for later retrieval.

2 Contraindications

No contraindications are currently known.


3 Safety instructions

Safety related tip

Do not touch the patient and the battery contacts simultaneously.

4 Operating elements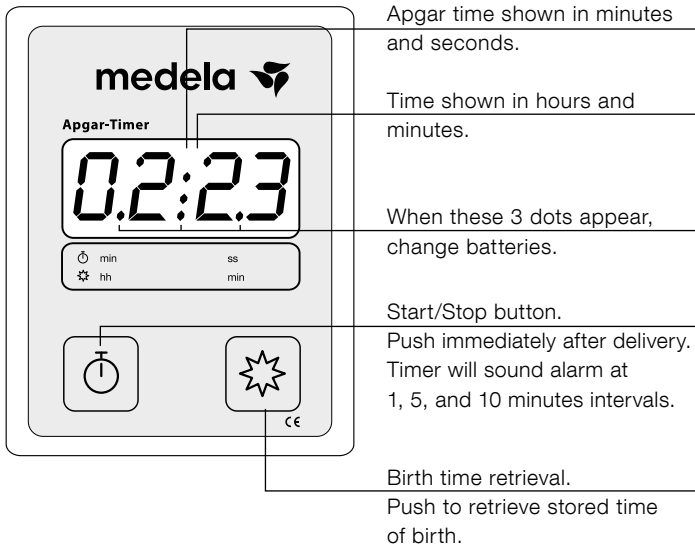

5 Operation of the Apgar Timer

1. Starting the Apgar Timer

Push the Start/Stop button.

2. Switching off the device

The Apgar Timer can be turned off by pushing the Start/Stop button. The Timer will automatically shut off after 1 hour.

3. Storing the birth time

When turning on the Apgar Timer the time of its internal clock is automatically stored as birth time. The birth time is shown in hours and minutes. The birth time will be changed as soon as the next Apgar score is taken.

4. Retrieval of the birth time

The birth time is called up in hours and minutes by pushing the birth time retrieval button.

5. Battery Check

A set of batteries should last for 1 000 Apgar score measurements. When the batteries become weak, 3 additional dots will appear on the display.

6. Change of Batteries

The Apgar Timer has been designed so that the internal clock continues to run for approx. 2 minutes when the batteries are changed. This means that when you change the batteries no adjustment of time will be necessary.

To change the batteries, proceed as follows:

1. Remove old batteries.
2. Insert new batteries (two 1,5 volt alkaline batteries) following the polarity markings.

7. Retrieval of the Internal Time

If you wish to check the time of the internal clock in the Apgar Timer proceed as follows:

If the Apgar Timer is running (Retrieval of the internal time is not available for the first ten minutes after the Apgar Timer is switched on.)

- Push the birth retrieval button twice. The Apgar Timer will return automatically to the timer after three seconds.

If the Apgar Timer is not running

- Push the birth retrieval button twice. Push the Start/Stop button to end showing the internal time.

8. Adjusting the time

The time of the internal clock can be set as follows:

1. Start with the Apgar Timer switched off.
2. Push the birth time and Start/Stop button simultaneously. A black dot will appear behind the first digit.
3. Push the birth time retrieval button until the correct number appears in the first digit.
4. Push the Start/Stop button, the black dot will jump to the second digit. Again push the birth time retrieval button until correct number appears in the second digit.
5. Follow the same steps to correct the third and fourth digit.
6. Leave the time adjusting mode by pushing the Start/Stop button. This will automatically save any changes done to the internal time.

6 Cleaning

General notes

The internal hospital directives take precedence. Wear suitable gloves for cleaning/disinfection.

- Switch off the device.
- The outside surfaces can be cleaned with a damp cloth.
- Cleaning or disinfecting agents should not be used in concentrated form!

Detailed information according to Medela Reprocessing Guide (product code 200.2391).

Surface cleaning materials

We recommend: – CaviWipes®, Metrex® Research,
www.metrex.com

– mikrozid® AF wipes, Schülke&Mayr GmbH,
www.schuelke.com

Do not use: Terralin (conc. 2%), Schülke & Mayr

7 Disposal

The Apgar Timer comprises metals and plastics and should be disposed of in accordance with the European directives 2011/65/EU and 2012/19/EU. Additional, local guidelines must also be observed. The electronic components and the batteries must be disposed of separately, in accordance with the local regulations. Please take care that you dispose the Apgar Timer and its accessories in accordance with your local disposal guidelines.

User information for the disposal of electrical and electronic equipment

This symbol means that the electrical and electronic equipment must not be disposed as normal household refuse. A correct disposal of this device protects and prevents possible damage to the environment or human health. For more information about the disposal contact the manufacturer, your local caregiver or healthcare provider. This symbol is only valid in the European Union. Please respect the relevant state laws and rules in your country for the disposal of electrical and electronic equipment.

8 Warranty

Medela AG warrants the device will be free from defects in materials and workmanship for a period of 2 years from the date of delivery. The manufacturer is not liable for any damage or consequential damage caused by incorrect operation, inappropriate usage as well as use by unauthorized persons.

9 Technical Data

| | |
|---------------|--|
| Batteries: | 2 pcs 1,5 V AA |
| Accuracy: | Quarzstability, <100 ppm |
| Signal: | Buzzing signal after 1, 5 and 10 minutes |
| Display: | 4-digit Liquid crystal display (LCD) |
| Operation: | 2 foil buttons |
| Measurements: | 160x100x35 mm (HxWxD) |
| Weight: | 250 g (including batteries) |

Operating conditions

| | |
|-------------------------|------------------|
| Temperature limitation: | +5 °C to +40 °C |
| Pressure limitation: | 70 kPa – 106 kPa |
| Humidity limitation: | 30 – 75% |

Transport/Storage conditions

| | |
|-------------------------|------------------|
| Temperature limitation: | -20 °C to +50 °C |
| Pressure limitation: | 70 kPa – 106 kPa |
| Humidity limitation: | 20 – 95% |

The Apgar Timer fulfils standard IEC 60601-1-2 where electromagnetic compatibility is concerned. While the Apgar Timer is in use, other equipment which also fulfils this standard will not be affected nor will the other equipment affect the operation of the Apgar Timer.

10 Signs and symbols

Start/Stop button and birth time memory. An alarm will sound after 1, 5 and 10 minutes.

This symbol indicates the compliance with the essential requirements of the Council Directive 93/42/EEC of 14 June 1993 concerning medical devices.

This symbol indicates the date of manufacture.

This symbol indicates to consult instructions for use.

This symbol indicates the manufacturer.

Birth time retrieval button.

This symbol indicates useful information about the safe use of the device.

This symbol indicates manufacturer's catalogue number.

This symbol indicates manufacturer's serial number.

This symbol indicates the temperature limitation for operation, transport and storage.

This symbol indicates the humidity limitation for operation, transport and storage.

This symbol indicates the atmospheric pressure limitation for operation, transport and storage.

This symbol indicates to not dispose the device together with unsorted municipal waste (for EU only).

This symbol indicates that the material is part of a recovery/recycling process.

This symbol indicates a carton package.

This symbol indicates to keep the device away from sunlight.

This symbol indicates to handle the fragile device with care.

This symbol indicates to keep the device dry.

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11 Technical Documentation

The Apgar Timer has been EMC-tested in conformity with the requirements of IEC 60601-1-2:2007 and IEC 60601-1-2:2014 4th Edition according to clause 7 and 8.9.

Electromagnetic compatibility (EMC, IEC 60601-1-2:2007 3rd Edition and IEC 60601-1-2: 2014 4th Edition, Table 1)

Electromagnetic Emissions

The Medela Apgar Timer is intended for use in the electromagnetic environment specified below. The customer or the user of the Medela Apgar Timer should assure that it is used in such environment.

| Emission Tests | Compliance | Electromagnetic environment - guidance |
|--|-------------------|---|
| RF emissions CISPR 11 | Group 1 | The Medela Apgar Timer uses RF energy only for their internal function. Therefore, its RF emissions is very low and is not likely to cause any interference in nearby electronic equipment. |
| RF emissions CISPR 11 | Class B | The Medela Apgar Timer is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes. |
| Harmonic emissions IEC 61000-3-2 | not applicable | |
| Voltage fluctuations / flicker emissions IEC 61000-3-3 | not applicable | |

Safety related tip

Wireless communication equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, and walkie-talkies can interfere with the Apgar Timer and should be kept at a minimum distance of 30 cm away from the Apgar Timer.

Electromagnetic compatibility (EMC, IEC 60601-1-2:2007 3rd Edition, Table 2)

| <h2 style="margin: 0;">Electromagnetic Immunity</h2> <p style="margin: 0;">The Medela Apgar Timer is intended for use in the electromagnetic environment specified below. The customer or the user of the Medela Apgar Timer should assure that they are used in such environment.</p> | | | |
|--|---|---|---|
| Immunity Tests | IEC 60601 test level | Compliance level | Electromagnetic environment - guidance |
| Electrostatic Discharge (ESD) IEC 61000-4-2 | ± 6 kV contact ± 8 kV air | ± 6 kV contact ± 8 kV air | Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %. |
| Electrical fast transient/burst IEC 61000-4-4 | ± 2 kV for power supply lines ± 1 kV for input / output lines | ± 2 kV for power supply lines ± 1 kV for input / output lines not applicable | Mains power quality should be that of a typical commercial or hospital environment. |
| Surge IEC 61000-4-5 | ± 1 kV differential mode ± 2 kV common mode | ± 1 kV differential mode ± 2 kV common mode not applicable | Mains power quality should be that of a typical commercial or hospital environment. |
| Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11 | <5 % UT (>95 % dip in UT) for 0.5 cycle 40 % UT (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles <5 % UT (>95 % dip in UT) for 5 sec | <5 % UT (>95 % dip in UT) for 0.5 cycle 40 % UT (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles <5 % UT (>95 % dip in UT) for 5 sec not applicable | Mains power quality should be that of a typical commercial or hospital environment. If the user of the electric breast pump Freestyle requires continued operation during power mains interruptions, it is recommended that the electric breast pump Freestyle is powered from an uninterruptible power supply or a battery. |
| Power frequency (50/60 Hz) magnetic field IEC 61000-4-8 | 3 A/m | 3 A/m | Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment. |
| NOTE UT is the a.c. mains voltage prior to application of the test level. | | | |

Electromagnetic Immunity

The Medela Apgar Timer is intended for use in the electromagnetic environment specified below. The customer or the user of the Medela Apgar Timer should assure that they are used in such environment.

| Immunity Tests | IEC 60601 test level | Compliance level | Electromagnetic environment - guidance |
|-------------------------------|-----------------------------|--------------------------|---|
| Conducted RF IEC 61000-4-6 | 3 Vrms 150 kHz to 80 MHz | 3 Vrms not applicable | <p>Portable and mobile RF communications equipment should be used no closer to any part of the Medela Apgar Timer, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = 1.2 \sqrt{P}$ $d = 1.2 \sqrt{P}$ <p>80 MHz to 800 MHz</p> $d = 2.3 \sqrt{P}$ <p>800 MHz to 2.5 GHz</p> <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m)</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey^a, should be less than the compliance level in each frequency range.^b</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p>  |
| Radiated RF IEC 61000-4-3 | 3 V/m 80 MHz to 2.5 GHz | 3 V/m | |

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

^a Field strengths from fixed RF transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Medela Apgar Timer is used exceeds the applicable RF compliance level above, the Medela Apgar Timer should be observed to verify normal operation. If abnormal operation is observed, additional measures may be necessary, such as reorienting or relocating the Medela Apgar Timer.

^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Electromagnetic compatibility (EMC, IEC 60601-1-2:2007 3rd Edition, Table 6)

Recommended separation distance between portable and mobile RF communications equipment and the Medela Apgar Timer

The Medela Apgar Timer are intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Medela Apgar Timer can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Medela Apgar Timer as recommended below, according to the maximum output power of the communications equipment.

| Rated maximum output power of transmitter W | Separation distance according to frequency of transmitter | | |
|---|---|--|--|
| | 150 kHz to 80 MHz $d = 1.2 \sqrt{P}$ | 80 MHz to 800 MHz $d = 0.35 \sqrt{P}$ | 800 MHz to 2.5 GHz $d = 0.7 \sqrt{P}$ |
| 0.01 | not applicable | 0.04 | 0.07 |
| 0.1 | not applicable | 0.11 | 0.22 |
| 1 | not applicable | 0.35 | 0.70 |
| 10 | not applicable | 1.11 | 2.21 |
| 100 | not applicable | 3.50 | 7.00 |

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Notes

Notes

International Sales

Medela AG
Lättichstrasse 4b
6341 Baar
Switzerland
Phone +41 41 562 51 51
Fax +41 41 562 51 00
ism@medela.ch
www.medela.com

Australia

Medela Australia Pty Ltd,
Medical Technology
3 Arco Lane
Heatherston, Vic 3202
Australia
Phone +61 3 9552 8600
Fax +61 3 9552 8699
contact@medela.com.au
www.medela.com.au

Canada

Medela Canada Inc.
4160 Sladeview Crescent Unit # 8
Mississauga, Ontario
L5L 0A1
Canada
Phone +1 905 608 7272
Fax +1 905 608 8720
info@medela.ca
www.medela.ca

China

Medela (Beijing)
Medical Technology Co., Ltd.
Unit M, 4F
No 1068 Wuzhong Road,
Minhang District
Shanghai 201103
P.R. of China
Phone +86 21 62368102
Fax +86 21 62368103
info@medela.cn
www.medela.cn

France

Medela France SAS
14, rue de la Butte Cordière
91154 Etampes cedex
France
Phone +33 1 69 16 10 30
Fax +33 1 69 16 10 32
info@medela.fr
www.medela.fr

Germany

Medela Medizintechnik
GmbH & Co. Handels KG
Postfach 1148
85378 Eching
Germany
Phone +49 89 31 97 59-0
Fax +49 89 31 97 59 99
info@medela.de
www.medela.de

Italy

Medela Italia Srl
Via Turrini, 13/15 – Loc. Bargellino
40012 Calderara di Reno (BO)
Italy
Phone +39 051 72 76 88
Fax +39 051 72 76 89
info@medela.it
www.medela.it

Japan

Medela K.K.
G-PLACE Sasazuka 13F
2-26-2, Sasazuka, Shibuya-ku
Tokyo 151-0073
Japan
Phone +81 3 3373 3454
Fax +81 3 3373 3457
info@medela.jp
www.medela.jp

Netherlands & Belgium

Medela Benelux BV
Uilenwaard 31
5236 WB's-Hertogenbosch
Netherlands
Phone +31 73 690 40 40
Fax +31 73 690 40 44
info@medela.nl
info@medela.be
www.medela.nl
www.medela.be

Poland

Medela Polska Sp. z o.o.
Ul. Lewinowska 8
03-684 Warszawa
Poland
Phone +48 22 865 12 50
Fax +48 22 864 38 65
info@medela.pl
www.medela.pl

Russia

OOO Medela
Vavilova str. 97, entrance 3
117335 Moscow
Russia
Phone +7 495 980 6194
Fax +7 495 980 6194
info@medela-russia.ru
www.medela-russia.ru

Spain & Portugal

Productos Medicinales Medela, S.L.
C/ Llacuna, 22 4a Planta
08005, Barcelona
Spain
Phone +34 93 320 59 69
Fax +34 93 320 55 31
info@medela.es
info@medela.pt
www.medela.es
www.medela.pt

Sweden

Medela Medical AB
Box 7266
187 14 Täby
Sweden
Phone +46 8 588 03 200
Fax +46 8 588 03 299
info@medela.se
www.medela.se

Switzerland

Medela AG
Lättichstrasse 7
6341 Baar
Switzerland
Phone +41 41 562 51 51
Fax +41 41 562 14 42
contact@medela.ch
www.medela.ch

United Kingdom

Medela UK Ltd.
Huntsman Drive
Northbank Industrial Park
Irlam, Manchester M44 5EG
UK
Phone +44 161 776 0400
Fax +44 161 776 0444
info@medela.co.uk
www.medela.co.uk

USA

Medela LLC
1101 Corporate Drive
McHenry, IL 60050
USA
Phone +1 877 735 1626
Fax +1 815 307 8942
info-healthcare@medela.com
www.medela-healthcare.us