SINGLE USE SILC CUP

SINGLE USE SILC CUP

EINWEG
SILC CUP SAUGGLOCKEVENTOUSE SILC CUP
A USAGE UNIQUE

SILC CUP MONOUSO

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This obstetrical instrument is utilized to assist a clinician in the delivery of an infant during childbirth.

Important note

Proper surgical procedures and techniques are the responsibility of the medical practitioner. Each physician must evaluate the appropriateness of the procedure based on his or her own medical training and experience.

General information

The Silc Cup is made of an elastomer and is available in one size, 60 mm inner diameter. The instrument is made in one piece and consists of a handle and a suction cup to be used with a pump and tubing. The cup has a smooth external surface, but there is a small longitudinal ridge, which makes it possible to observe any rotation. The inside of the cup is lined with small projections. The projections enable the air between the cup and the child's head to be evacuated and ensure that the cup sucks firmly against the child's head. The soft suction cup deforms, thereby ensuring optimal adhesion and minimising trauma to the fetal scalp.

Pump requirements

- This obstetrical instrument has been developed for use in conjunction with an external vacuum source capable of precise vacuum regulation to ensure proper vacuum levels (up to -80 kPa/-600mmHg).
- The vacuum source should be designed for medical purposes and may be either electric or manual.
- We recommend using the Medela Basic suction pump.
- Note:** Wall suction is not to be used.

Tubing requirements

The tubing connecting the instrument to the vacuum source is vital for safe and proper function of the instrument. To ensure this only Medela original tubing should be used.

Safety rules

- Do not use if:
 - The product has exceeded the expiration date on the packaging.
 - Packaging is opened or damaged.
 - If the device is damaged.
 - If the device shows clear safety defects.
 - Wall suction is not used.
 - Vacuum extraction must only be performed or supervised by trained and experienced clinicians.
 - The neonatal care staff must be aware of the indications, contraindications and precautions for assisted delivery and must watch for the signs of complications of vacuum assisted delivery.
 - Operative vaginal delivery should not be performed when the probability of success is very low.
 - Ensure against intrusion of extraneous tissues and ensure proper placement of cup.
 - Attention: Never apply the cup to any portion of the infant's face as this can cause fetal trauma.
 - Insertion should be performed carefully to avoid malfunctions, maternal or fetal trauma and maternal discomfort.
 - The head, just not the scalp, must descend with each pull.
 - Pull only during a contraction.
 - Always apply traction at an angle allowing the stem of the cup to remain in-line or on-axis.
 - Never apply rocking movements or torque to the device; only steady traction should be used.
 - Discontinue vacuum extraction if there is evidence of significant scalp trauma or damage.
 - The head must be delivered, or almost completely delivered, with no more than three pulls.
 - The head must be completely delivered within 15–20 minutes of first applying the cup.
 - Never exceed 10 minutes of cumulative traction time.
 - Never exceed maximum recommended vacuum -80 kPa (-600 mmHg) or "pop offs" (2) – scalp trauma can occur.
 - Discontinue vacuum extraction if the vacuum cup becomes disengaged or pops off twice.
 - Do not pull three times, if the second pull fails to produce good descent.
 - Parents should be instructed to check the baby's health.
 - The infant has to be watched for several days for possible problems.

Disposal

Single use only. After use, dispose of according to local regulations.

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

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